



Millennium M10 Operating Instructions

Warning: Do not use extension cords or electrical adapters

1. Select a location that allows the oxygen concentrator to draw in room air without being restricted. Make sure that the oxygen concentrator is at least $\frac{1}{4}$ ft. away from the walls, furniture, and especially curtains that could impede adequate airflow to the device. Keep away from heat.
2. Plug the power cord into an electrical outlet.
3. Connect the oxygen tubing to the oxygen outlet port.
 - i. You may also attach a humidification cup to the oxygen outlet port. Make sure the cup is filled to recommended amount and attach cannula to humidification cup.
4. Press the power switch to the On (I) position. Initially all the indicator lights will turn on and the audible alarm will sound for a few seconds. After that time, only the green indicator light should remain lit.
5. After turning on the Oxygen Concentrator, you can begin breathing from the device immediately; however, allow at least 10 minutes for oxygen delivery to reach defined specifications.

6. Adjust the flow to the prescribed setting by turning the knob on the top of the flow meter until the ball is centered on the line marking the specific flow rate.
7. Be sure oxygen is flowing through the cannula. One way to check is to put the nose piece in a cup of clean water and see bubbles. If it is not, refer to troubleshooting section.
8. Put the cannula on as directed by your home care provider.
9. When not using the device, press the power switch to the Off (O) position.